


Annalise Enterprise CXR

The evolution of diagnostic-support AI for chest X-ray

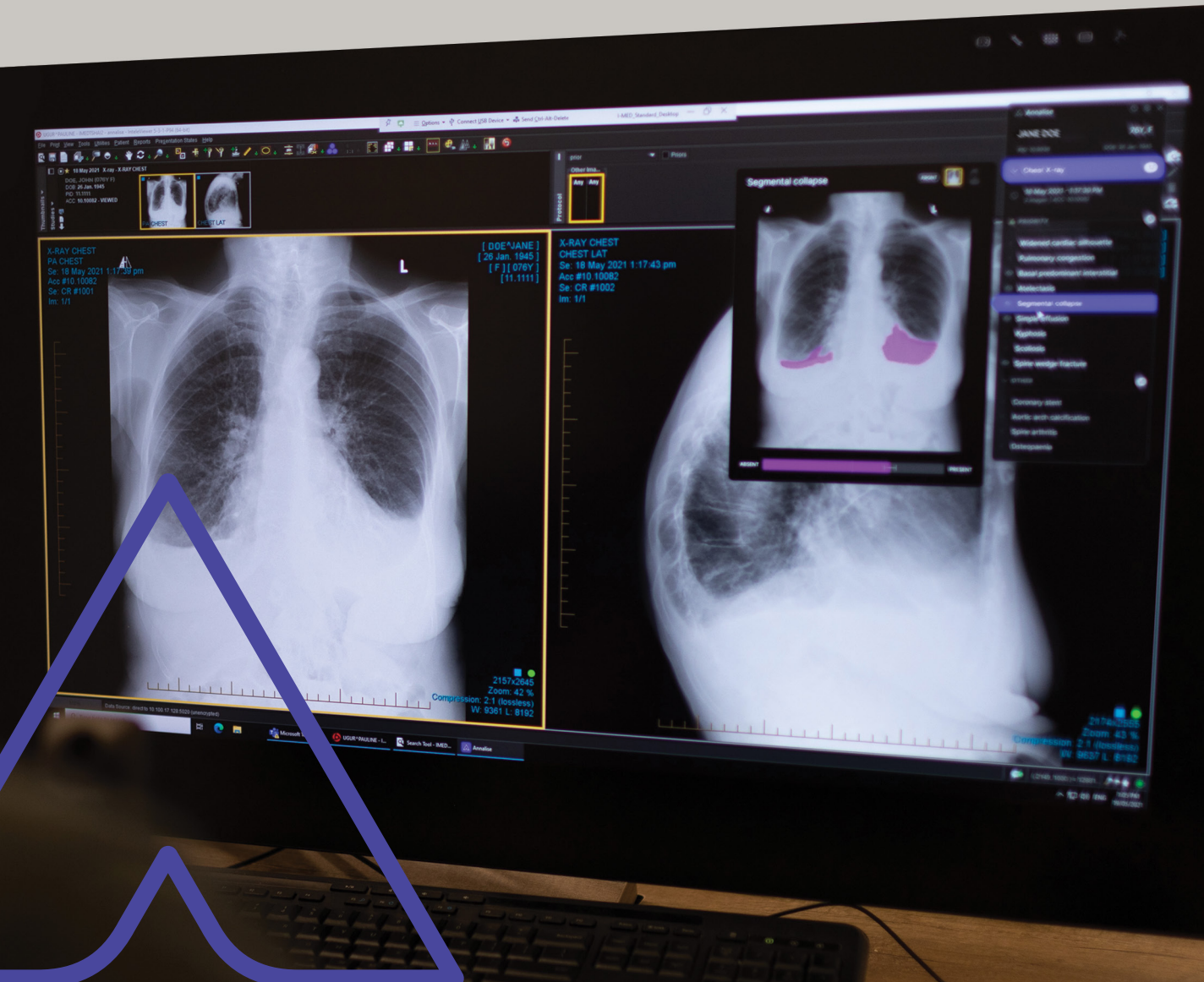

This is not just a simple pneumothorax tool. This is not just a pleural effusion tool.

This is a comprehensive decision-support solution for chest radiographs capable of detecting up to 124 findings.

Designed by clinicians for clinicians.



1. **A second pair of eyes:** a reassuring second-read that never tires, for clinicians dealing with fatigue, frequent interruptions and increasing workloads.



2. **Worklist prioritisation:** AI-enhanced triage, designed to optimise the reporting workflow with on configurable priority grouping based on the AI results, such as flagging cases you deem critical or cases you deem unremarkable.*



3. **Improve efficiency:** report cases faster, saving valuable time that can be spent elsewhere in the clinical workflow, such as additional examinations, enhancing the interaction with patients, etc.



4. **Improve accuracy:** Radiologists assisted by the tool on average perform better than unassisted radiologists in detecting present findings.

*Specific features may not be available in all regions.

See the chest like never before.

Annalise Enterprise CXR can identify the suspected presence of up to 124 findings, highlights the localisation of specific findings and is intelligently designed to help users review AI results quickly.



- 124**
 - 1. Detect up to 124 findings:** including pneumothorax, lung nodules, pneumomediastinum, fractures (ribs, clavicle, humerus), nasogastric tube positioning, and many more.
 - 2. Lateral and frontal views:** view the findings in lateral and frontal views (PA or AP), analysing up to three images per study.*
 - 3. Identify the location of findings:** localisation of certain findings are highlighted in the Viewer. Localisation can be toggled on and off.*
 - 4. Interpret with confidence:** the confidence bar displays the likelihood of the finding and uncertainty of the AI model, assisting clinicians to interpret with confidence.

* Available with the Annalise Viewer.


Detailed results. Powered by clinically robust AI.

The size, quality and diversity of training data are essential to developing a high-performing and generalisable AI model. Hand-annotating each chest X-ray study for a comprehensive number of findings also improves overall accuracy by providing extra findings for training that could otherwise be confused with a single finding.

Trained on one of the world's largest radiologist hand-annotated chest X-ray datasets.

Data diversity

Dataset derived from inpatient and outpatient settings, gathered from three continents. Data was captured by devices from various manufacturers, including FujiFilm, Canon, Carestream, Kodak, Philips, Siemens, and others.

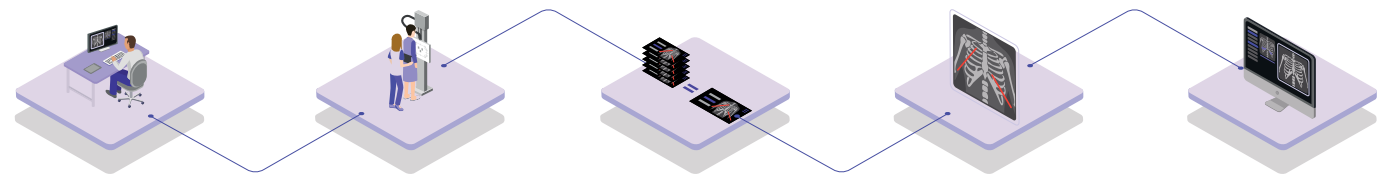

Trained by

145+ consultant radiologists trained in standardised annotating procedures to ensure a consistent and high-standard of annotations across all data-sets.

From a data-set of

782,000+ unique CXR studies.

Labelled by

at least **3** radiologists, who independently hand-annotated each CXR study.

Generating

280,000,000+ CXR annotations.

To bring you

124 findings detected by Annalise Enterprise CXR.

Annalise Enterprise CXR improves radiologist accuracy when used as an assist tool

Multi-reader multi-case validation study design summary



1. 20 radiologists took part in the study



2. All with substantial clinical experience after radiology specialist training (median = 10.5 years)




3. Each interpreted all 2,568 cases with and without Annalise Enterprise CXR after a 3-month washout period



4. Cases were ground truthed by x3 specialist thoracic radiologists

AI-assisted vs. unassisted radiologist accuracy

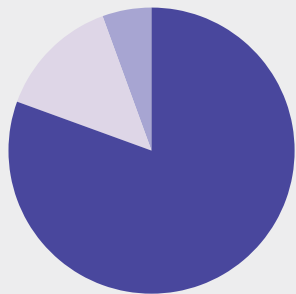


■ superior ■ non-inferior ■ inconclusive

Reporting accuracy is non-inferior in 95% (n=121) of all clinical findings and delivers better diagnostic accuracy in 80% (n = 102) of clinical findings.

Significant performance improvements


Accuracy Improvement

(Macro average AUC across all findings, starting at 0.5)

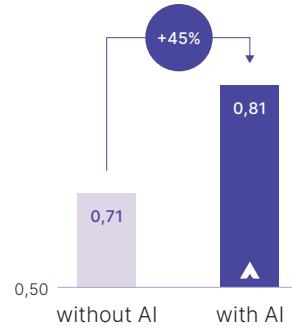


Efficiency improvement

(Interpretation time per case, seconds)

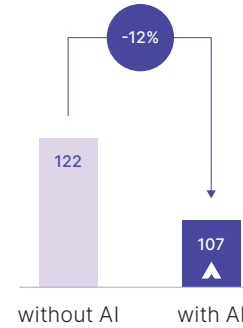


Significant finding improvements

Simple Pneumothorax (AUC)

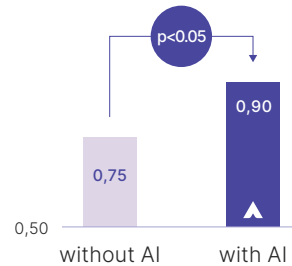

Rib Lesion (AUC)

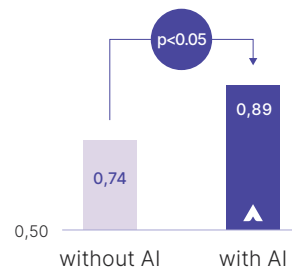

The Annalise Enterprise advantage

Annalise Enterprise is an enterprise IT solution offering hospitals and radiology providers access to Annalise.ai's comprehensive AI modules, including CXR and CTB, intended to assist clinicians with the interpretation of radiological imaging studies.



One platform for comprehensive AI

Annalise Enterprise seamlessly integrates directly within RIS/PACS and can be launched from either platform. With infrastructure in place, Enterprise partners are the first in line to simply expand AI capabilities with subsequent comprehensive AI modules as they're released.



Worklist Triage*

Annalise Enterprise's AI models analyse studies as they're acquired and provide a notification of urgent cases in your site's reporting worklist. With the comprehensive number of findings and integration with your PACS, RIS or reporting worklist system, radiologists can more efficiently prioritise their work and report on the cases they deem most urgent first.



Consistent user interface experience

Radiologists appreciate the similar user experience and Annalise Viewer Interface across Annalise Enterprise, saving time and reducing frustration learning new solutions. Secondary capture output is also available.

*Specific features may not be available in all regions. Check current regulatory status with an Annalise.ai employee
Source: Seah, JCY et al.; Lancet Digit Health 2021; 3: e496-506

Radiological findings covered by Annalise Enterprise CXR

Technical Factors

Patient rotation
Cervical flexion
Underinflation
Underexposed
Overexposed
Incompletely imaged chest
Image obscured

Lines and Tubes

In position central line (CVC)
In position endotracheal tube (ETT)
In position nasogastric tube (NGT)
In position pulmonary arterial catheter (PAC)
Intercostal drain
Suboptimal central line (CVC)
Suboptimal endotracheal tube (ETT)
Suboptimal nasogastric tube (NGT)
Suboptimal pulmonary arterial catheter (PAC)

Cardiac Devices

Electronic cardiac devices
Cardiac valve prosthesis
Sternotomy wires

Orthopaedic Implants

Rib fixation
Shoulder fixation
Shoulder replacement
Rotator cuff anchor
Clavicle fixation
Spinal fixation

Scapula

Scapular fracture

Cardiomediastinum

Widened cardiac silhouette
Inferior mediastinal mass
Superior mediastinal mass
Hilar lymphadenopathy
Calcified hilar lymphadenopathy
Pneumomediastinum
Unfolded aorta
Widened aortic contour
Aortic arch calcification
Pulmonary congestion
Pulmonary artery enlargement
Pericardial fat pad

Airspace Opacity

Focal airspace opacity
Multifocal airspace opacity
Diffuse lower airspace opacity
Diffuse upper airspace opacity
Perihilar airspace opacity
Diffuse airspace opacity

Interstitial

Upper zone fibrotic volume loss
Lower zone fibrotic volume loss
Upper interstitial thickening
Basal interstitial thickening
Diffuse interstitial thickening
Diffuse fibrotic volume loss
Diffuse nodular / miliary lesions
Upper zone bullae
Lower zone bullae
Diffuse bullae

Pulmonary Lesion

Solitary lung nodule
Solitary lung mass
Multiple masses or nodules
Cavitating mass(es)
Cavitating mass with content
Calcified granuloma (< 5mm)
Calcified mass (> 5mm)
Nipple shadow

Airways

Reduced lung markings
Peribronchial cuffing
Hyperinflation
Bronchiectasis
Tracheal deviation

Pneumothorax

Simple pneumothorax
Tension pneumothorax

Pleural Thickening

Calcified pleural plaques
Pleural mass
Diffuse pleural thickening

Collapse

Atelectasis
Segmental collapse
Lung collapse
Post resection volume loss

Non-surgical Foreign Body

Internal foreign body

Abdomen

Subdiaphragmatic gas
Distended bowel
Hiatus Hernia
Gallstones
Gastric band
Suboptimal gastric band

Clavicle

Acute clavicle fracture
Chronic clavicle fracture

Humerus

Acute humerus fracture
Chronic humerus fracture
Shoulder dislocation
Shoulder arthritis

Diaphragmatic Contour

Diaphragmatic elevation
Diaphragmatic eventration

Spine

Kyphosis
Scoliosis
Spinal wedge fracture
Spinal arthritis
Diffuse spinal osteophytes
Osteopaenia

Chest Wall Contour

Pectus carinatum
Pectus excavatum

Soft Tissues

Subcutaneous emphysema
Mastectomy
Breast implant
Calcified axillary nodes
Calcified neck nodes

Bone Lesion

Spinal lesion
Scapular lesion
Humeral lesion
Rib lesion
Clavicle lesion

Surgical Clips and Stents

Mediastinal clips
Neck clips
Axillary clips
Abdominal clips
Lung sutures
Aortic stent
Coronary stent
Airway stent
Oesophageal stent
Biliary stent

Ribs

Acute rib fracture
Chronic rib fracture
Rib resection

Pleural Effusion

Simple effusion
Loculated effusion

This information is intended for health care professionals only. This device is not intended to provide direct diagnosis. For detailed device information, including indications for use, contraindications, and warnings, please consult the user guide prior to use.

Annalise Enterprise CXR is not for sale in the US. Not all features are available in all regions, check current regulatory status with an Annalise.ai employee. info@annalise.ai

Address: Annalise-AI Pty Ltd., Level P, 24 Campbell Street, Sydney NSW 2000, Australia, ABN: 92 635 645 260

Annalise.ai, ANNALISE ENTERPRISE and the Annalise logo are trademarks of Annalise-AI Pty Ltd, registered in Australia and other countries and regions..

Annalise Enterprise CXR is a class IIb device under Regulation (EU) 2017/745

OPT-PRM-056 Ver 5

